

CE Dent Dry

PRIME DENTAL'S DENT DRY is a liquid formulation ideally suitable for degreasing and drying of dentin. **DENT DRY** can be used before any filling material. Root Canal Sealants and dentin bonds may not flow properly in presence of excess moisture and even traces of oil / grease, **DENT DRY** is highly useful in such situation.

INDICATION : Drying of cavity before bonding agents are applied. Drying of root canal before sealants are applied.

CONTRA INDICATION : Do not use on soft tissue.

PRECAUTIONS : Do not mix with water or other liquid before using. Close Cap of the bottle tightly. Keep away from open flame.

APPLICATION : With help of cotton pellets soak **DENT DRY** dentin surface, which is to be dried and degreased. Wait for few seconds till applied liquid evaporates than continue with work. Complete evaporation of the applied liquid is very important before any filling material are applied. **DENT DRY** shall remove excess moisture but shall not desiccate the dentin surface.

PACKING : Bottle of 50 ml

STORE : Store in cool dark place. Keep away from direct flame.