

Product Description: RC Cal-I is a formulation of calcium hydroxide, Iodoform with barium sulphate in ready to use paste form. Calcium Hydroxide has been proved as one of the most efficient material for various applications in dentistry. RC Cal-I is highly alkaline radiopaque water soluble paste which could be easily cleaned and removed from canal whenever required.

Indications:

- Leaking canal, Apexification, Temporary root canal filling material.

Contra Indications: No known contra indication exists for product.

Precautions: Do not mix with water while using, close cap of the syringe immediately to avoid hardening of the material.

Application: After preparation of root canal, cleaning and drying, push the paste into the canal while gradually withdrawing needle from the canal (give gentle pressure on plunger). Remove excess with cotton swab and seal off with cement. As calcium hydroxide is a resorbable material if any periapical extrusion has happened the same should not give any problem. While removing RC Cal-I from canal, use K File with few drops of water.

Storage:

- Store away from sunlight in cool and dry place. Keep away from direct flame.

Package contents:

- Syringe of 2g, 5 application needles.

Note: The material has been developed for professional dental use and it is to be used by a dental professional only. Application should be carried out strictly according to the instruction for use. User is responsible for testing the material for its suitability. The manufacturer cannot be held liable for damages resulting from failure to observe the indication and procedure of use as mentioned in the instructions for use. The user should not use material for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.