

Prior to use carefully read the  
Instruction For Use

**Product Description:**

Prime's Primecresol is a solution for root canal cleansing and disinfection. Due to its antibacterial activity, Primecresol is efficient in vital pulpotomy and for the treatment of severely infected root canals.

**Components:** Cresol, Formaldehyde, Glycerine and Distilled Water.

**Indication:**

Root Canal Cleansing and dressing

**Contra Indications:**

It should not be used with patients who have a history of severe allergic reaction to any of the components.

**Precautions :** Avoid contact with skin and eyes.

**Warning:**

Contains formaldehyde which can be toxic by inhalation or ingestion. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

**Procedure:**

1. Place a cotton wool soaked in Primecresol and remove excess liquid before placing into the pulp chamber and seal.
2. Remove the cotton wool dressing after a week time and restore the tooth.

**Storage & Shelf Life:**

Store away from sunlight in a cool and dry place. Keep away from direct flame. Do not leave the bottle open. Do not use after expiration date.

**Disposal Conditions:**

To be disposed of based on the prevailing rules of the country in which it is used.

**Presentation:**

Primecresol: 15ml x 1