

Product Description: Caries Indicator liquid contains organic dye in a glycol base that stain carious dentin red. Helps to identify demineralized dentin.

Indication:

To identify demineralized dentine. Identify caries in difficult to see places. Locate the root canal orifice.

Contra Indication:

No known contra indication is recorded.

Precaution:

Avoid splattering. Stain is removed from carious areas mechanically. If dye is difficult to remove (intraorally) use a sodium hypochlorite solution. If it stains clothing, then wash clothing with detergent and warm water or use white vinegar

Application:

Paint Caries Indicator liquid on the surface to be checked. Leave it for 10 seconds then rinse and vacuum.

When Caries Indicator is used to identify root canals in the early stages of endodontic procedure, a small red dot will remain at the opening of the canal orifice.

For caries removal use a slow speed round bur or excavator. Repeat as needed until caries is completely removed.

Storage: Store away from sunlight in cool and dry place.

Package contents

• 15ml Bottle