

Product Description: TMP-RS is a temporary filling materials self-curing under humidity for temporary filling of cavities. The final hardness of both variants decreases in the sequence of listing. TMP-RS can be removed with help of probe and excavator.

Indications:

- TMP-RS is regular setting.

Applications:

- Use an instrument to fill the required quantity into the wet cavity. The hardening process starts after a few minutes. Avoid any exposure to chewing pressure for about 2 hours after application.
- Deep cavities need not be filled completely. If the material is used to close up a drug, pressure-free application is recommended.

Note:

- Due to sensitivity to moisture, close the jar immediately after removing the material, since otherwise the viscosity of the material increases (due to entry of moisture), and the material may no longer be usable.

Storage and Shelf-Life:

- Store away from sunlight in cool & dark place.
- Do not use after the expiration date.

Package contents:

- 40g Bottle.