

**Product Description:** Image is an Eugenol free impression paste for edentulous cases, hence shall not irritate oral tissue and lips of the patient while taking impression.

**Indications:** For taking impression for edentulous cases and relining of dentures.

**Mixing:** Remove equal length of both paste, approximate 10cm length for maxillary and 8cm length for mandibular impressions. Mix well with the help of broad blade spatula for 30 seconds, till uniform orange colour is achieved. Load special trays [shellac or acrylic] immediately so as to have even thickness of material all around.

**Impression:** Insert tray in patient's mouth. Complete border molding within two and a half minutes from start to mix. Wait for further two and a half minutes before tray is removed from patient's mouth. Wash thoroughly under running water.

**Timing:**

- Mixing: 30 seconds.
- Loading: Upto 60 seconds from start of mix.
- Border molding: Upto 2 minutes 30 seconds from start to mix.
- Withdraw: 4 minutes 30 seconds to 5 minutes from start of mix.
- Model casting: 1 hour from withdrawal from patient's mouth.
- Remove cast: Immediate on final setting of stone or die stone plaster.

**Model Preparation:** No separating medium is required while pouring impression, however Prime Dental's Debubbler spray can avoid surface bubbles. We recommend to wait for 1 hour before model is casted. The impression is readily removed from the model after immersion in hot water at 70°C

**Storage :** Store away from sunlight in cool and dry place.

**Package Contents :**

125g Base and 85g Catalyst.