

Product Description : Prime Dental Modelling Wax pink sheets is specially formulated for use in Dentistry technique.

Made of pure, natural bees wax and resins, so that it softens easily under hot water. Easy to carve, suitable for a variety of Dental techniques.

Composition : Beas Wax, Paraffin Wax, Microcrystalline Wax, Mineral oil and Colorant.

Indications : As an articulating medium.

For preparing bite block for occlusal registration.

For preparing spacer for custom tray.

Sheets of pink wax for modelling dentures bases and acrylic saddle. Making wax pattern for complete and partial dentures.

Making wax pattern of Orthodontic appliances.

Instruction for Use : Before use plastify the sheets of wax by dipping it in warm water or holding over the flame of a burner. It is easy to mould when softened and not tear to flakes.

Indirect technique : Wax pattern is made on cast/die in lab outside the oral cavity.

Contraindications : It is contraindicated for use in patient who have a history of severe allergic reaction to any of the ingredient. **Storage Conditions** : Store in a cool and dry environment between 10°C to 24°C. **Shelf life** : 3 years from date of manufacture.

Presentation :Types 12 Sheets in 225gm mono pack.

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated indications and methods. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.