

Prior to use carefully read the
Instruction For Use

Prime CMCP

Camphorated Parachlorophenol

Active Root Canal Disinfectant

Product Description

Prime CMCP is a antibacterial solution for the disinfection of the root canal system. Its composition ensures to remain active and effective over an extended period.

Composition: Para chlorophenol and Camphor.

Intended Use: Antimicrobial treatment and disinfection of the dental pulp chamber and root canals.

Indications

Endodontic Dressing: Disinfection following pulpectomy and treatment of post-traumatic inflammation.

Canal Sterilization: General disinfection of the root canal system.

Application Instructions:

Soak a sterile cotton pellet in the solution.

Place the pellet into the pulp chamber and seal the cavity with a temporary restoration, ensure no leakage or leaching of product in oral cavity.

Leave the dressing in place for not more than one week.

During follow up visit, remove the cotton pellet and proceed with the permanent restoration.

Important: To prevent cross-contamination, never return dispensed material to the original container.

Single-use application only.

Safety Warnings and precautions:

Skin Contact: Always work with protective gloves accidental direct contact may cause irritation, chemical burns, or hypersensitivity.

First Aid: If skin contact occurs, immediately wipe the area thoroughly with a cotton swab soaked in alcohol, then wash the skin well with soap and water.

Allergic Reactions: If a rash, sensitization, or any other allergic reaction develops, stop using the product immediately and seek medical attention.

Eye Contact: Always work with protective glass / eye shield, if accidentally the product enters the eyes, rinse them thoroughly with fresh water and immediately consult a physician.

Contraindications: Do not use this product on patients with a known history of severe allergic reactions to any of its ingredients.

Store: Always Store Temperature between 10°C to 25°C away from direct light, moisture in cool place Always Store in cool and dry place away from direct light.

Shelf life: 3 years from the date of manufacturing.

Disposal: To be disposed as per the prevailing rules of the country in which it is used.

Presentation : 1 x 15ml Plastic Dropper Bottle

***Note:** The material has been developed for professional dental use and it is to be used by a dental professional only. Application should be carried out strictly according to the instruction for use. User is responsible for testing the material for its suitability. The manufacturer cannot be held liable for damages resulting from failure to observe the indication and procedure of use as mentioned in the instructions for use. The user should not use material for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.*