

Product description:

Prime Dental Products offers unique needles

(having size 30 gauge) for root canal irrigation with 2 side vents. Side vents shall avoid irrigation fluid to be pushed apically hence an ideal choice while irrigation. We offer 2 side vents instead of conventionally 1 side vent, further reducing pressure. Both vents are designed to situate opposite each other offering more radial wash. As needle is blocked from front chances of irrigation fluid being pushed apically is further reduced.

Instruction of use :

Gently eject irrigation fluid like Sodium Hypochlorite

(3% & 5.25% with mint aroma) or distilled water or chloro hexidine solution in the canal. Sterilize needle before use.

Precaution :

Twents are not for injection purpose. Do not lock Twents inside the canal while irrigation.

Storage:

Store away from sunlight in cool and dry place.

Packing :

» 10 non- sterile needles.

Prime Dental Products also offers following Endo Products :

- | | |
|--|--|
| » Sodium Hypochlorite 3% & 5.25%: With mint aroma | » RC Pex : Calcium Hydroxide Paste with Iodoform |
| » RC Help : Lubricating EDTA Paste | » RC Solve : GP & ZnOE solvent |
| » RC Cal : Calcium Hydroxide Paste | » RC Seal : Epoxy Bonded Root Canal Sealer |
| » Prime MTA Bioceramic Root Canal Sealer | |