

Prior to use carefully read the
Instruction For Use

Product Description: Retra Liquid is a formulation of aqueous aluminium chloride clear solution which is an excellent agent for retraction of gingival. Ferric sulphate & ferric sub-sulphate solution for retraction shall retard setting of silicone impression material while Retra Liquid shall not retard them. Aluminium Chloride also has good astringent property.

Indications:

- Retraction of gingival during crown & Bridge procedure. Whenever Astringent action is required.

Contra Indications:

- No known contra indication is recorded.

Precautions:

- Avoid application to soft tissue where retraction or astringent action is not Desired.
- Shall not stain cloth.

Application:

- After selecting desired size of retraction cord dip cord into Retra Liquid for 2 minutes and use. Retraction action of the cord shall be enhanced Substantially.

Storage:

- Store in cool and dry place.

Package contents:

- Bottle of 30ml.

Note: The material has been developed for professional dental use and it is to be used by a dental professional only. Application should be carried out strictly according to the instruction for use. User is responsible for testing the material for its suitability. The manufacturer cannot be held liable for damages resulting from failure to observe the indication and procedure of use as mentioned in the instructions for use. The user should not use material for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.