

Zinc Oxide Eugenol Quick Set Cement Polymer Reinforced Intermediate Restorative Material

Indications:

- Intermediate / Temporary seal for medicament dressings.
- Insulating cavity lining allowing the immediate placement of amalgam restorations

Mixing: Place an appropriate amount of powder and liquid onto a clean glass slab. Incorporate successive small portions of powder into the liquid ensuring complete mixing after each addition. Continue adding powder until a cohesive consistency is reached.

Mixing time: 1-1½ minutes.

Working time: 2 minutes from end of mix.

Set time: (3 minutes 30 seconds) to (4 minutes 30 seconds)
[depends on ambient room temperature and glass slab temperature.]

Insertion: Place the cement in the dried clean cavity and gently pack into position. To prevent the cement sticking to the instrument during placement dip the tip into the surplus powder on the mixing slab or use small pellet of cotton wool dampened in water and hold in a pair of pointed tweezers.

Finishing: Once set, approximately 3 minutes after insertion in the mouth - the cement can be trimmed and finished. In the case of cavity linings, clear the margins prior to placement of the filling material.

Special Notes: This product is formulated for dental use only. Avoid contact with eyes. In case of contact, rinse eyes immediately with plenty of water and seek medical attention. Replace cap tightly after use.

Important: Do not use cements as base for composite restorations as this may cause softening of the overlaying composite material.

Storage: Store away from sunlight in cool and dry place.

Package contents: 15g Powder & 10ml Liquid.