

Product Description: Dent Retract is a formulation of aqueous ferric chloride clear solution which is an excellent agent for retraction of gingival. Ferric chloride solution for retraction shall retard setting of silicone impression material while Dent Retract shall not retard them. Ferric Chloride also has astringent property.

Indications:

- Retraction of gingival during crown & Bridge procedure. Whenever Astringent action is required.

Contra Indications:

- No known contra indication is recorded.

Precautions:

- Avoid application to soft tissue where retraction or astringent action is not Desired.
- Shall stain cloth.

Application:

- After selecting desired size of retraction cord dip cord into DENT RETRACT For 2 minutes and use. Retraction action of the cord shall be enhanced Substantially.

Storage:

- Store in cool and dry place.

Package contents:

- Bottle of 30ml.