

Product Description: Debubbler is fast drying surface tension reducer and cleanser; it can be used on wax before pouring of the investment and also on all types on impression materials before casting models. This is available in pleasant blue liquid in a spray bottle.

On wax pattern: Spray the patterns at around 3 inch distance taking precaution that material does not slump. air blow the pattern to remove any slump. However there is no need to wait till pattern is completely dry, it could be invested when it appears moist. Debubbler is compatible to all types of investment and does not retard setting of investment.

On impression: Method of application remains as above with special precaution to be taken that the liquid which is accumulated in fine details area, air blow to remove any accumulation. This material is compatible to all types of impression and it does not retard setting of stone or die stone plaster, hence other procedures shall remain same.

Storage:

- Store away from sunlight in cool and dry place.
- Keep Away from direct flame.

Package contents:

- Spray bottle of 100ml.