

Product Description:

Restorite Ionomer Flow is a glass ionomer powder based single component, light cured cement. It is intended for use as a lining material, in extended fissure sealing and smaller lesions.

Indications:

1. A lining cement used under all kinds of filling materials
2. Extended fissure sealing
3. Restoration of smaller lesions.

Contra Indications:

The product is contraindicated for use in patients with known allergies to methacrylate-based composites or fluoro alumino silicate glass.

Precautions:

1. Avoid prolonged or repeated contact with skin (allergic contact dermatitis). If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention
2. Avoid contact with eyes. If the product comes into contact with eyes, rinse thoroughly with fresh water and consult a physician.
3. Avoid prolonged inhalation and do not ingest the material.

Procedure:

1. Thoroughly clean and air dry the cavity.
2. Dispense the material using an applicator tip into the cavity directly.
3. Do not exceed the thickness of 1mm.
4. Light cure using standard curing light for 30 to 40 seconds and then proceed with standard restoration technique.

Storage & Shelf Life:

Store away from sunlight in a cool and dry place. Keep away from direct flame. Do not leave the syringe open. Always screw cap tightly back on to the syringe after each use. Do not use after expiration date. Shelf life is 3 years from the date of manufacturing.

Disposal Conditions:

To be disposed of based on the prevailing rules of the country in which it is used.

Presentation:

2g Syringe x 5 Capillary Tips