

Product Description: RC Solve is a liquid for gutta percha & zinc oxide cement dissolution. Liquid is derivative of orange oil which shall not irritate soft tissue or periapical area, also offers pleasant orange fragrance. It does not evaporate as rapidly as chloroform or other solvents.

Indication: During repeat of root canal treatment, GP Points or Zinc Oxide cement is desired to be removed from canal. While placing post partial removal of GP Points is desired.

Method of use: Apply few drops of the liquid on top of the canal from which GP Points or Zinc Oxide cement is desired to be removed, wait for 3 to 4 minutes and use Gates Glidden drills or Pecho Reamers for removal of GP points. In curved canal hand instruments should be used.

Contra Indication: No known contra indication of the product has been reported

Note: RC Solve should not be used at or beyond the apex of the canal.

Storage: Store away from sunlight in cool and dry place.

Package contents: 15ml Glass Bottle.