

Product Description: Templute is an eugenol free temporary luting cement, which is easy to work with. It is to be used for temporary cementation of crowns, bridges and permanent inlays. It could also be used above medicinal material in cavity. This material shall not retard or hamper setting of composite filling or luting agents, not it shall hamper bonding materials.

Indications: Temporary cementation of crowns, bridges & inlays.
Temporary filling over medicinal agents

Contra Indication: Permanent cementation of crowns, bridges & inlays.
Permanent filling material. No systemic side effects are known.

Application: Templute is to be used at approx 25°C temperature. Lower temperature would retard setting and higher would shorten working time. Product is balanced to work at equal volume of the paste extruded on mixing pad or glass slab. Both paste are mixed with clean and dry cement spatula [plastic or stainless steel] for 30 seconds, moving in circular motion to achieve homogeneous consistency. Apply thin layer of mixed mass inside temporary crown or bridge and seat in place with gentle pressure. For cementing inlay small amount of mixed mass can be placed in cavity also. It is desirable to clean & dry tooth surface before cementation. Wet tooth surface shall reduce effect of luting. Templute offers working time of 3 minutes after mixing and initial setting is completed remove excess cement with help of sharp instrument like explorer or stickle scaler.

Storage: Store away from sunlight in cool and dark place. Keep away from direct flame.

Package contents: 50g Base (White), 15g Catalyst (Lemon Yellow) & Accessories.